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Instructions for use

Select diffusion time by using ◯ and ⊖ buttons. Press the ⊕ button to start. The screen shows the diffusion time and count down starts. At the end of the diffusion cycle, automatic bleeding starts.

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Email : gestion@hygiasept.com - Site : www.hygiasept.com

Active Hygiene of air and surfaces by micro diffusion

220V EASYSEPT®

DESCRIPTION & ADVANTAGES OF THE DIFFUSER.

Easysept is a professional portable diffuser for dry micro-diffusion and can be carried easily in all premises to be treated. It enables aerobiodecontamination, disinfection (**according to NF T 72-281 Standard 2009 version with Directed Manual Process**), odour destruction and disinsectisation.

Easysept is equipped with a demountable flexible and a multi-directional head for a targeted treatment. It is then possible to treat mattresses, curtains, hangings and not easily accessible places. This function is especially used to destroy insects and to regularly treat air-conditioning ducts and air shafts, bath inserts : the favourite destinations for bad odours and undesirable insects.

The conception of Easysept is very elaborated:

It is equipped with an automatic cleaning system which evacuates every liquid leftover at the end of the cycle before the compressor stops and a load miser allows a constant discharge and an homogeneous formation of fog.

| SPECIFICATIONS | |
|---|--------------------|
| Power : | 120 W |
| Outlet pressure (air) : | 3,5 bar |
| Flexible length: | 1,20 m |
| Bodywork type : | Portative |
| Bodywork material : | Painted Aluminium |
| Overall length : | 35 cm |
| Overall width : | 23 cm |
| Overall height : | 36 cm |
| Weight empty / Overall Weight : | 7,7 kg / 8,4 kg |
| Power supply : | 230 Vac |
| Replacement Container : | 2 litres container |
| Delrin nozzle and titanium injector – Liquid circuit automatic valve system at the end of each diffusion. | |

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Instructions for use

Select diffusion time by using (⊖) and (⊕) buttons.
Press the (S) button to start.
The screen shows the diffusion time and count down starts.
At the end of the diffusion cycle, automatic bleeding starts.

www.hygiasept.com

L'Hygiène Active !

HYGIASEPT® diffusers are guaranteed for one year.

The use of products outside the Hygiasept® range are not recommended with this diffuser and if used, we take no responsibility.

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